

<b>Parameter Increased</b>	<b>TIME</b>	<b>SNR</b>	<b>CNR</b>	<b>RESOLUTION</b>	<b>ARTIFACTS AND NOTES</b>
TR	+	+	-	N/A	Increased TR reduces T1 weighting
TE	N/A	-	+	N/A	Increased TE increases T2 weighting
ETL	-	-	+	N/A	High ETL can cause image blur; increases T2 weighting; MT effects; reduces susceptibility
BANDWIDTH	-	-	-	N/A	Reduces echo spacing; increase image sharpness; decrease chem shift; reduce motion
FREQ MATRIX	+	-	N/A	+	Increases TE
PHASE MATRIX	+	-	N/A	+	
FOV	N/A	++	N/A	-	Reduces wrap artifact
NEX	++	+	N/A	N/A	Reduces motion; SNR increase sqrt2
SLICE THICKNESS	N/A	++	N/A	-	Increases volume averaging

From: <https://wiki.virtual-scan.com/> - Knowledge Portal

Permanent link: [https://wiki.virtual-scan.com/doku.php?id=library:mri\\_parameter\\_relationships&rev=1737495366](https://wiki.virtual-scan.com/doku.php?id=library:mri_parameter_relationships&rev=1737495366)

Last update: 2025/01/21 21:36

